TUMOR İŞARETLEME İĞNESİ

KULLANMA KLAVUZU

TR

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INSTRUCTIONS FOR USE:

ENG

PROFID Tumor Marking needle is a single lumen needle with ergonomic handle and with pre-loaded fiducial gold markers at distal tip.

Indications For Use: It is a needle for placement of gold seed before radiation treatment and/or pre-surgery localization of soft tissue tumors. Gold tumor markers are implanted in the tumor tissue to act as a radiologic landmark for ensuring accurate target localization of tumors during radiation therapy.

Contraindications: Biopsy / aspiration sets or needles should be used by a physician trained in interventional techniques and familiar with the possible side effects, typical findings, limitations, indications and contraindications of biopsy procedures. Physician judgement is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

Caution:

Ethylene oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Store in dry place, under room temperature, protect from sun light. Possible allergic reactions should be considered. It must be used only in hospitals by physicians who familiar with the device and procedures. For single Patient Use Only. Do not attempt to clean or resterilize this product. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

LOCALIZATION PROCEDURE:

1. Identify the localization site preferably under Ultrasound or CT.
2. Using the sterile technique, prepare the skin with antiseptic and drape.
3. If necessary, infiltrate the selected area with local anaesthetic.
4. For prostate seeding, place the Transrectal probe to right position.
5. Open the needle pouch and remove protective sheath from the needle.
6. Insert the Tumor marking needle (*for prostate seeding through Ultrasound probe*) and advance into the targeted area.
7. Set the Tumor marking needle in the lesion.
8. Once satisfied with location of needle tip, please remove the safety pin.
9. Place gold tumor markers into lesion as shown in Fig.1 and Fig.2
10. Remove the needle.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

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MARGE MEDİKAL DAN.İTH.İHR.SAN.VE TİC.LTD.STİ